Babyplan® Fertility Gel

Instructions for Use/Bruksanvisning/Brugervejledning/Käyttöohje/Gebruikershandleiding/Manuel d'utilisation/Instrucciones de uso

Intended Use/Avsedd användning/Tiltenkt bruk/Tilsligtet anvendelse/Käyttötarkoitus/Beoogd gebruik/Usage prévu/Usso previsto

EN: This fertility gel helps sperm move more easily toward the egg, which may increase the chances of pregnancy. It also helps keep sperm healthy by creating a supportive environment where they can survive longer.

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EN: Fertility Gel is applied inside the vagina before the sexual intercourse. Dispose of the packaging adequately after use.

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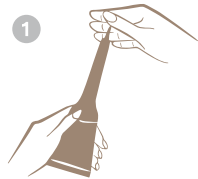
EN: Break the tip of the tube coming out of the bag by bending the thin part sideways.

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EN: Place the tip of the tubing on inside of the vagina as shown and gently squeeze the bottom of the tube.

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EN: Fertility gel is formulated to provide a balance of pH to increase the survival of the sperm and to increase the survival. It contains magnesium and calcium ions to support conception.

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EN: Fertility Gel supports the motility of the sperm as it reaches the egg, thus increasing the probability of fertilization. In addition, its content is effective in maintaining its viability as it simulates the environment in which the sperm can survive.

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EN: Infection in the genital area. Irritated genital area. Known allergic condition to any of its ingredients. Breastfeeding status. Excessive vaginal fluid secretion.

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EN: Keep between 5 °C and 30 °C. Keep away from direct sunlight. Fertility is not a contraceptive gel product. Do not use as a vaginal lubricant. Avoid contact with eyes. Do not swallow. Use the entire tube, do not store it for second use. Do your general personal cleaning after the application and sexual intercourse. Keep away from children. Store in dry and cool places. Do not be used in those who are allergic to the ingredients. Do not be used after curettage.

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EN: There is no known side effect when used in accordance with the instructions for use and warnings.

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	Storage temperature		Expiry date (month/year)		Medical device
	Keep in a dry place		Keep away from sunlight		Reference number
	Caution		For single use only		Unique Device Identification (UDI)
	Follow instructions for use		Manufacturer	# icon" data-bbox="430 919 460 941"/>	Model number
	Lot number		Date of manufacture (month/year)		
	Non-sterile		Do not use if package is damaged		

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E-Invasion A/S